

# Arthroscopy Knee | Instruction for Use


## Included in the Setup

- 2102 Short Knee Positioner
- 2101 Knee Skin Right
- 2202 C-Clamp (2)
- 2100 Sports Med Knee Right

## Accessories

- 2111 Sports Med Instrument Set
- 2103 Demo 30° Arthroscope

**Tactile**  
ORTHOPAEDICS

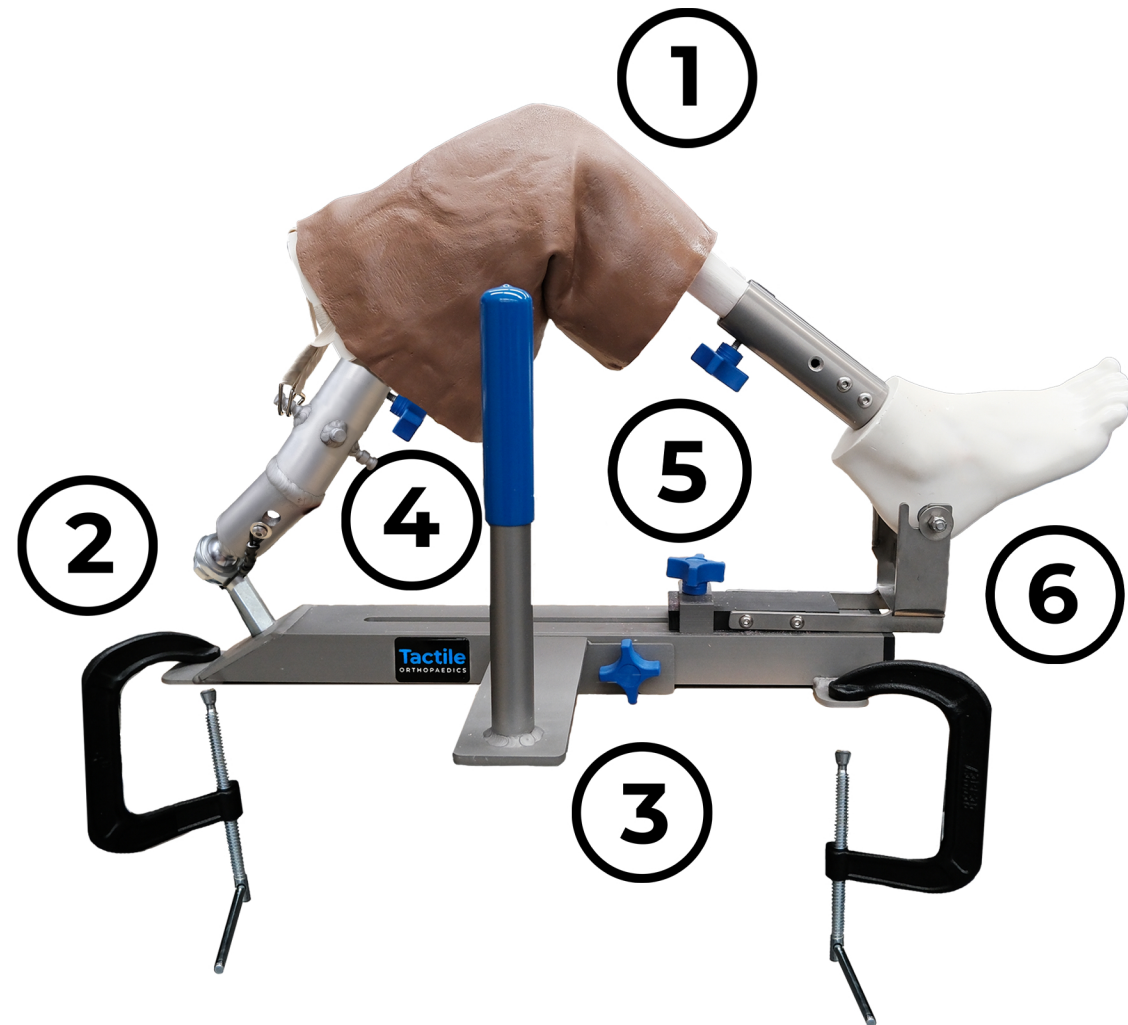

## How to use:

1. Pull the skin sleeve over the knee model from the tibia side and insert the the foam on the femoral side;
2. Affix the positioner to a table using the C-clamps provided;
3. Fasten the valgus post using the blue thumbscrew;
4. Insert the femur side of the knee model into the femoral tube, aligning the indicator marks, and secure with the thumbscrew;
5. Put the tibia side of the model into the tibial tube with the attached foot, aligning the indicator marks, and secure with the thumbscrew;
6. Place the foot into the saddles.

Not a Medical Device

[tactile-ortho.com](http://tactile-ortho.com)

Made in Canada