

Arthroplasty Knee | Instructions for Use


Included in the Setup

- 2002 Long Knee Positioner Right
- 2101 Knee Skin Right
- 2202 C-Clamp (2)
- 2000 Arthritic Varus Knee Right

Accessories

- 2006 Fan
- 2004 Arthritic Instrument Set
- 2108 Vacuum

Tactile
ORTHOPAEDICS

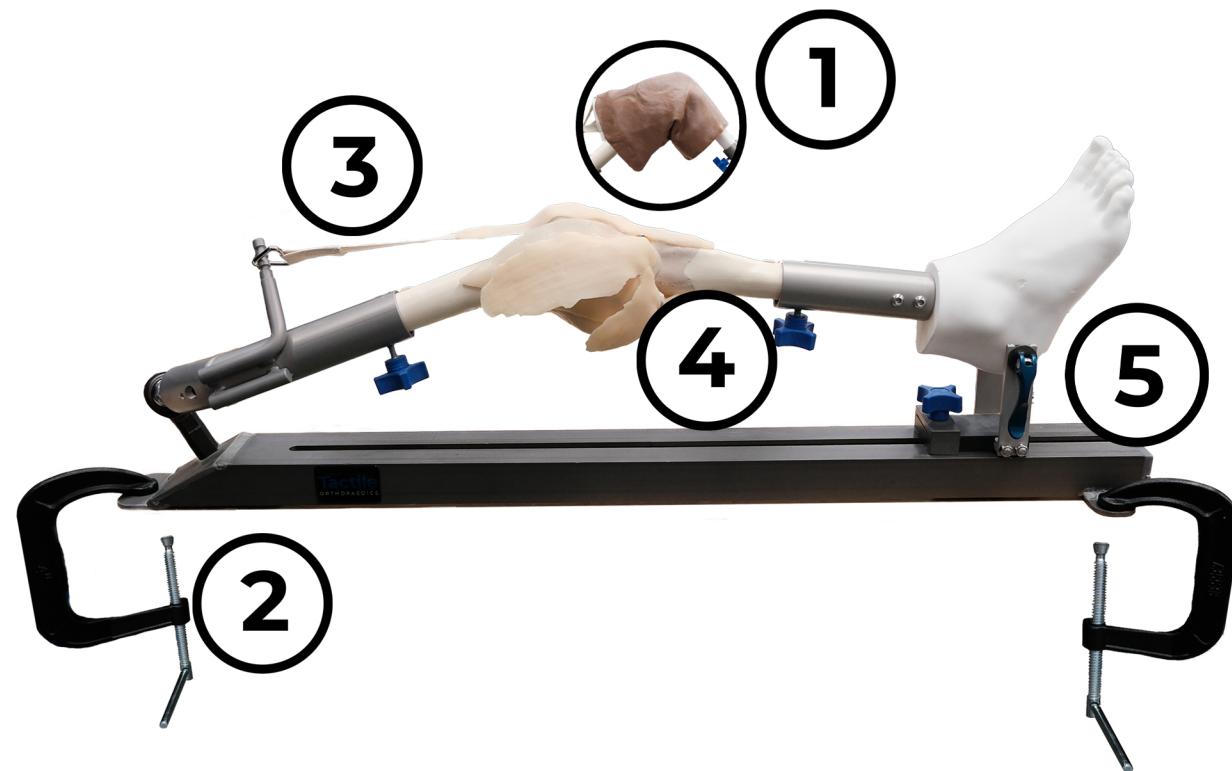

How to use:

1. Pull the skin sleeve (if using) over the knee model from the tibia side and insert the the foam on the femoral side;
2. Affix the positioner to a table using the C-clamps provided;
3. Insert the femur side of the knee model into the femoral tube, aligning the indicator marks, and secure with the thumbscrew;
4. Put the tibia side of the model into the tibial tube with the attached foot, aligning the indicator marks, and secure with the thumbscrew;
5. Place the foot into the saddles and tighten the blue quick release.

Not a Medical Device

tactile-ortho.com

Made in Canada