

# Arthroscopy Shoulder | Lateral Decubitus | Instructions for Use


**Tactile**  
ORTHOPAEDICS

## Included in the Setup

- 2302 Sports Med Shoulder Positioner
- 2301 Shoulder Skin Right
- 2202 C-Clamp
- 2300 Sports Med Shoulder Right

## Accessories

- 2103 30° Demo Arthroscope
- 2108 Vacuum (Debris Management)
- 2111 Sports Med Instrument Set

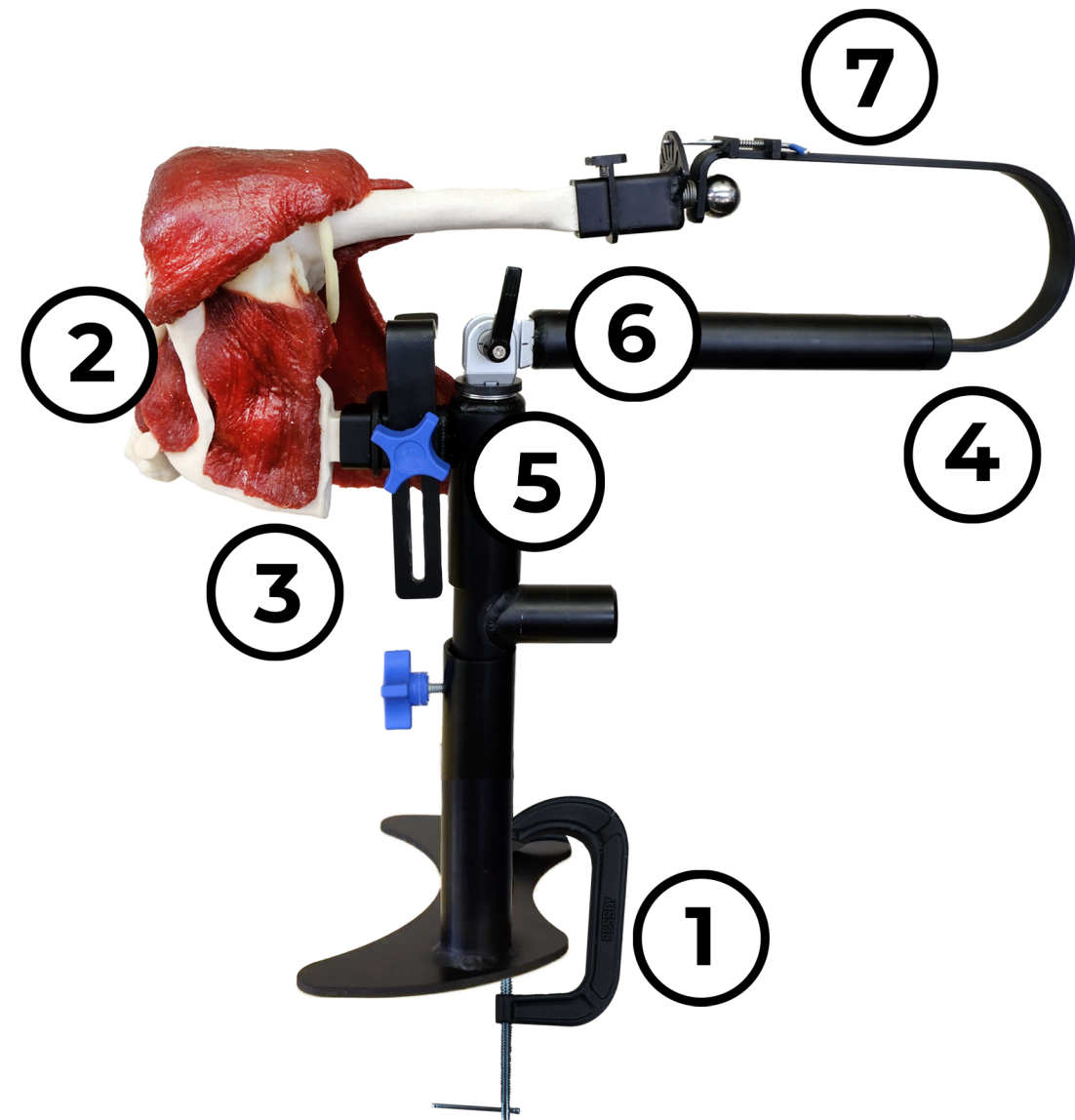

## How to use:

1. Affix the positioner to a table using C-clamp provided;
2. Ensure the shoulder positioner is in the lateral decubitus orientation as shown in the photo;
3. Press the quick connect button and insert the scapula side of the shoulder model into the shoulder positioner. Release the quick the quick connect button. Pull on the model to ensure it is secure in the positioner;
4. Move the slider arm into position, press the quick connect button and insert the humerus into the shoulder positioner. Release the quick connect button. Pull on the slider arm if traction is desired;
5. If lateral distraction is desired, loosen the blue thumbscrew to slide the lateral traction post into the desired position and tighten the thumbscrew;
6. If abduction is desired, release the locking handle, set accordingly and lock the handle in place.
7. Pull the sliding pin to adjust rotation. Lock in place when desired rotation is achieved.

Not a Medical Device

[tactile-ortho.com](http://tactile-ortho.com)

Made in Canada